



Clinic | Care | Homecare | Assist

The durable care bed for

safety and comfort



## The durable care bed for patient comfort

Comfort and safety for the occupant, alleviation for care staff and an unique design – the Classiko unites many strengths to offer good value for money. This **durable care bed** has the advantage of robust design and a range of adjustment options that make every-day life more agreeable.

The four-section mattress base adapts to suit many different needs. For example, by raising the back and thigh rests and tilting the bed into the optional reverse Trendelenburg position, the occupant is moved into a comfortable sitting position for eating or watching television. With a height adjustment range from 40 to 80 cm, the Classiko makes getting in and out of bed easy and safe while ensuring **an ergonomic working position** for care staff.

The motorised model of the bed can be operated effortlessly with a **selectively lockable handset.**  An electrically adjustable bed is not necessary in all situations. Stiegelmeyer therefore also offers the Classiko in a **mechanical version.** By dispensing with cables and manually removable elements, this model meets the **safety requirements in psychiatric care** particularly well. With a high safe working load of 225 kg, it is also suitable for heavier occupants.

Care staff will find the mechanical Classiko easy to operate. Height adjustments via the footpedal with the help of a **hydraulic pump** can be made just as easily as the force-assisted adjustments of the back and thigh rests. The Classiko offers visual appeal with a large selection of different **head and footboards** and **wood decors** to suit individual requirements.









## Optimum protection for the occupant

The safety sides on the Classiko can be adjusted to meet the individual protection needs of the occupant. The standard version of the bed is equipped with **fulllength safety sides** on both sides of the mattress base for optimum safety (Figure 1).

If the occupant's mobility needs to be supported, then the **combinable half-length safety sides** are a good choice (Figure 3). When raised at the head end only, enough room is left for getting in and out of bed. The split safety side can also be combined with a fulllength safety side on the other side of the bed.

Independent occupants with good mobility will not need safety sides at all – in this case, the Classiko can be fitted with a **fixed side facing**, which lends the bed a particularly homelike style (large picture).

With its equipment and accessory options, the Classiko is a comfortable bed that can be adapted to meet individual requirements. The four-section mattress base with steel wire mesh is robust and flexible (Figure 2). In the motorised model, the **auto-contour setting** and the **sliding backrest** help protect the occupant from shear forces.

For taller patients, we recommend **bed extensions** to 210 or 220 cm (Figure 4). Matching mattress extension elements and extended safety side bars are also available.









The Classiko is easy to move and manoeuvre. Thanks to its **centrall locking castors**, care staff can lock or unlock all four castors at once, by pressing one brake pedal (Figure 5). The motorised model is operated by use of a **handset**, which shows all of the options as easily understandable pictograms (Figure 6). Individual functions can be blocked for the occupant's safety, if required.

# Technical data

Dimensions and weights	Classiko	
External dimensions (depending on the wooden surround)	approx. 208 x 101 cm	
Mattress base (mattress dimensions)	200 x 90 cm	
Backrest length compensation	approx. 10 cm	
Safe working load	175 kg electrical	
	225 kg mechanical	
Ground clearance	approx. 15 cm	
Total weight	approx. 130 kg	

## Dimensions and adjustment ranges



	5
Lowest mattress	Highest
base position: 40 cm	base pa

Ľ			
ttress			
~~			

. position

Orthopaedia






Raised leg position

up to approx. 1



208 to 212 cm

200 cm

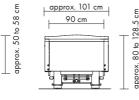

# Safety and comfort for psychiatric care

The mechanical model of the Classiko is perfectly suited to the high safety requirements in psychiatric care. Since it has **no cables** and its components are protected against removal, it prevents patient injuries and damage to the bed. At the same time, the comfort and **homelike appearance** of the Classiko enhances patient wellbeing.

For further information on use in psychiatric care, go to www.stiegelmeyer.com

# Equipment and options

### **Standard features**

### (mechanically adjustable)

- Four-section mattress base
- Mattress base with steel wire mesh base
- Ball-bearing castors Ø 125 mm
- Permanently unlockable castors
- Four centrally locking castors
- Height adjustment range 40 to 80 cm
- Backrest adjustment to 70°
- Flat, raised leg position to 11 °
- Two-bar full-length safety sides
- A wide range of wooden surrounds in various decor combinations
- Mechanically adjustable
- Argentum metal finish
- Safe working load 225 kg

### **Standard features**

### (electronically adjustable)

- Four-section mattress base
- Mattress base with steel wire mesh base
- Ball-bearing castors Ø 125 mm
- Permanently unlockable castors
- Four centrally locking castors
- Height adjustment range 40 to 80 cm
- Flat, raised leg position to 11 °

## Decors

The Classiko can be adapted to suit any room with a variety of attractive decors to choose from. The selection ranges from the lighter Lindberg Oak to the striking dark Bella Noce Choco. Our customer service would be happy to show you the Stiegelmeyer colours from our latest collection.

- Auto-contour setting
- DBfK backrest
- Handset with selective locking function
- Two-bar full-length safety sides
- A wide range of wooden surrounds in various decor combinations
- Adjustable with electric motor
- Argentum metal finish

### **Optional equipment**

- Tilting to reverse-Trendelenburg position
- Three-bar full-length safety sides
- Integral castors Ø 125 mm
- Double castors Ø 150 mm
- Tilting to Trendelenburg and/or reverse Trendelenburg position
- Bed extension 210 or 220 cm
- Linen holder
- Wall deflection rollers, horizontal
- Wall deflection rollers, vertical, at head end
- Fixed side facing, instead of safety sides
- Sleeve for attaching universal holder
- Combinable safety sides, split on one or both sides



### Ihr Bett – unsere Leidenschaft

Your bed - our passion

#### Finland / Suomi

Stiegelmeyer Oy Ahjokatu 4 A 15800 Lahti Phone +358 (0) 3 7536 - 320 Fax +358 (0) 3 7536 - 330 E-Mail stiegelmeyer@stiegelmeyer.fi finland.stiegelmeyer.com

### Netherlands / Nederland

Stiegelmeyer BV Peppelenbos 7 6662 WB Elst Phone +31 (0) 345 581 - 881 E-Mail info@stiegelmeyer.nl netherlands.stiegelmeyer.com

#### Poland / Polska

 Stiegelmeyer Sp. z o.o.

 86-212 Stolno, Grubno 63

 Phone
 +48 56 6771 - 400

 Fax
 +48 56 6771 - 410

 E-Mail
 info.stolno@stiegelmeyer.pl

 poland.stiegelmeyer.com

#### France

Stiegelmeyer SAS Bâtiment B 27 chemin des Peupliers 69570 Dardilly Phone +33 (0) 4 7238 - 5336 Fax +33 (0) 4 7238 - 5481 E-Mail contact@stiegelmeyer.fr france.stiegelmeyer.com

#### South Africa

Stiegelmeyer Africa (PTY) Ltd The Estuaries, Century City, Cape Town, 7441 Phone +27 (0) 21 555 - 1415 Fax +27 (0) 21 555 - 0639 E-Mail info.southafrica@stiegelmeyer.com south-africa.stiegelmeyer.com



 Stiegelmeyer GmbH & Co. KG

 Ackerstraße 42, 32051 Herford, Germany

 Phone
 +49 (0) 5221 185 - 0

 Fax
 +49 (0) 5221 185 - 252

 E-Mail
 info@stiegelmeyer.com

 www.stiegelmeyer.com